


1



Unpack your LOOP Learning System CPR Controller (LOOP Controller). Do not plug it in yet!

2

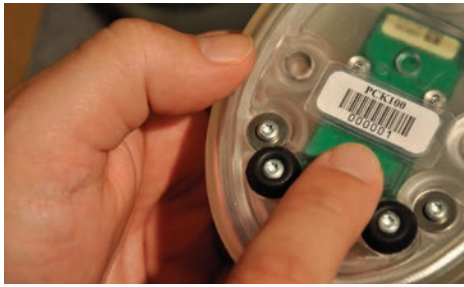

Locate the serial number on the back of your LOOP Controller.

3




Go to LOOPcpr.com. Click "Register LOOP" and enter the serial number.

4



Download the LOOP Learning System CPR Software installer.

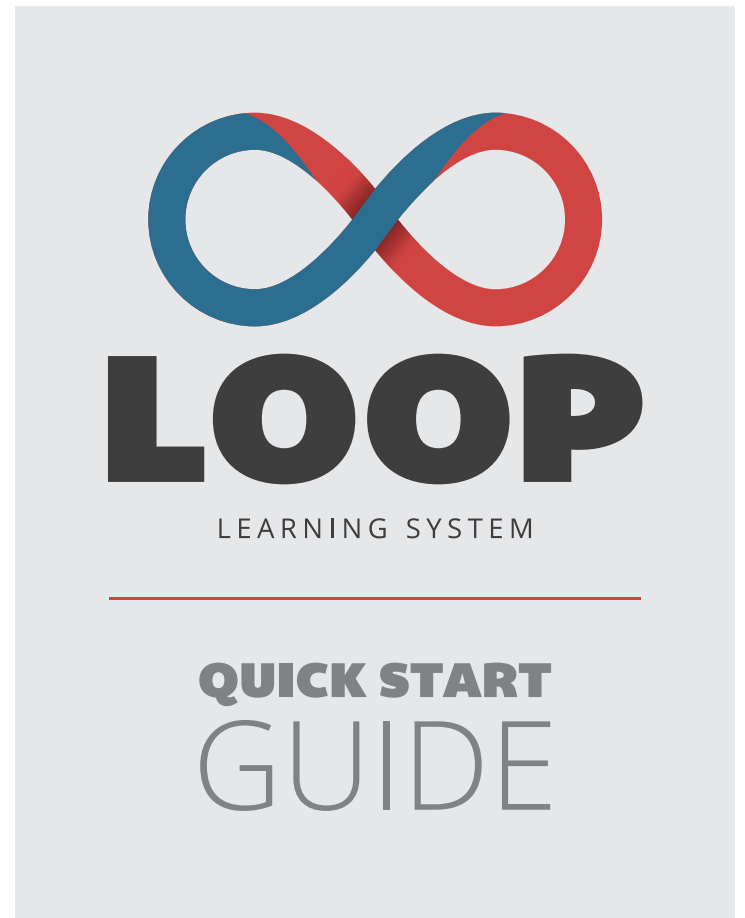

5



Once the installer has downloaded, find and run the installer.

6



Plug your LOOP Controller into an active USB port and start the LOOP Learning System Software!

HOW-TO AMPLIFY YOUR CPR SKILLS PRACTICE PROGRAM

BRING YOUR STUDENTS TO LIFE WITH THE LOOP LEARNING SYSTEM

2 PRIMARY COMPONENTS

1

LOOP CPR CONTROLLER

Think of the Controller as your CPR measurement hub - tracking depth of compressions, rate of compressions, how well ventilations were performed and more!

MICRO SWITCH

Determines if the controller is under compression.

GYROSCOPE

Determines 3D orientation and position.



ACCELEROMETER

Senses motion and acceleration.

2

LOOP CPR SOFTWARE



INTERACTIVE

Game play with graphics, music and video.



REALTIME METRICS

Immediate measurement and feedback to improve technique.



SCORECARD

Results and data provided to you and your students.

