

## Welch Allyn OAE Hearing Screener Clinical Competency Evaluation

**Purpose:** This evaluation certifies that \_\_\_\_\_ has met Welch Allyn's requirements for utilizing the OAE Hearing Screener. The training involved hands-on exercises. Although further training may be needed, this individual has demonstrated competency in the following areas at this time.

TASK	Performed Task	Needed Training	Did Not Observe
1) Properly plugs in equipment	_____	_____	_____
2) Properly turns on equipment	_____	_____	_____
3) Powers on label printer	_____	_____	_____
4) Demonstrates how to clean probe	_____	_____	_____
5) Follows facility infection control	_____	_____	_____
6) Launches Screener program	_____	_____	_____
7) Records patient Log Number	_____	_____	_____
8) Swaddles baby and/or prepares patient for test	_____	_____	_____
9) Selects patient's ear in software	_____	_____	_____
10) Selects proper sized probe tip	_____	_____	_____
11) Places probe into patient's ear	_____	_____	_____
12) Troubleshoots probe fit if needed	_____	_____	_____
13) Knows how to View Details	_____	_____	_____
14) Prints results from label printer	_____	_____	_____
15) Restarts for next patient	_____	_____	_____
16) Provides appropriate counseling to patient or caregiver regarding results	_____	_____	_____
17) Documents results according to facility's protocol	_____	_____	_____

The above evaluation was successfully completed on \_\_\_\_\_.

\_\_\_\_\_  
 \_\_\_\_\_

Name and Title of Person Supervising the Competency Evaluation