

SURGICAL MASK WITH PROCEDURE EARLOOPS-PRODUCT DATA SHEET

Product ID Code	3900001
Product Name	Surgical Mask
Product Description	3-ply, with filter (BFE> 98%) and procedure ear loops, integrated, moldable nose piece, designed for single use
TEST RESULTS	
ASTM F2101	Bacterial Filtration Efficiency -BFE > 98%
ASTM F1862	Resistance of Medical Face Masks to Penetration by Synthetic Blood - Passed at 120 mm Hg
MIL-M-36954C	Differential Pressure - Passed at 3.9 H ₂ O/cm ²
16 CFR PART 1610	Flammability - Passed Class 1
PRODUCT INFORMATION	
Material	Outer layer (colored) polypropylene - hydrophobic, middle layer polypropylene melt-blown, inner layer (white) polypropylene - hydrophilic, nose piece - wire with polyethylene coating, ear loops - polyester / elastane
Restriction of Use	The user is obliged to check the suitability of the article for the intended purposes if necessary
Allergy Notice	Not made from natural rubber latex, Fiberglass free
Storage	Store at room temperature in a dry and ventilated place
Standard Size	6.875" W x 3.75" H
Color	Blue outer layer, White inner layer
Packing Unit	50 Masks per Box
Carton	Contains 50 Boxes - Totaling 2500 Masks
Palette	Contains 25 Cartons - Totaling 62,500 Masks
Country of Origin	United States of America

INSTRUCTIONS FOR USE



1. WASH YOUR HANDS BEFORE TOUCHING A MASK




2. ENSURE THE PROPER SIDE OF THE MASK FACES OUTWARDS



3. LOCATE THE METALLIC STRIP AND PLACE IT ON THE NOSE BRIDGE



4. SECURE THE STRINGS BEHIND YOUR HEAD OR OVER YOUR EARS

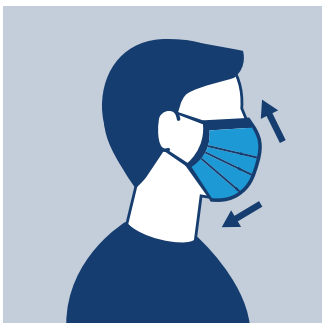

5. COVER MOUTH AND NOSE FULLY MAKING SURE THERE ARE NO GAPS



6. PRESS THE METALLIC STRIP TO FIT THE SHAPE OF THE NOSE



7. REMOVE THE MASK FROM BEHIND BY HOLDING THE STRINGS WITH CLEAN HANDS



8. DISPOSE THE MASK IN A CLOSED BIN WITHOUT TOUCHING THE FRONT



9. SURGICAL MASK PLACED CORRECTLY



10. DO NOT TOUCH THE MASK WHILE USING IT, IF YOU DO, WASH YOUR HANDS



11. REPLACE THE MASK IF IT GETS DAMP AND DO NOT REUSE IT