

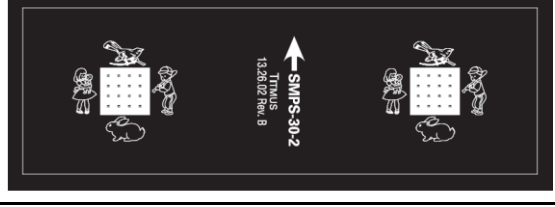
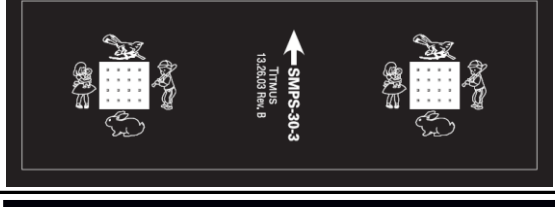
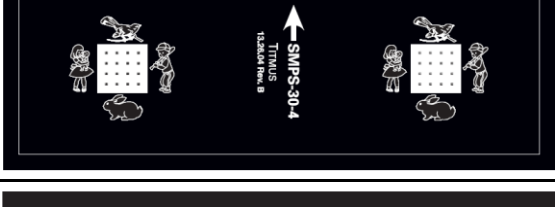

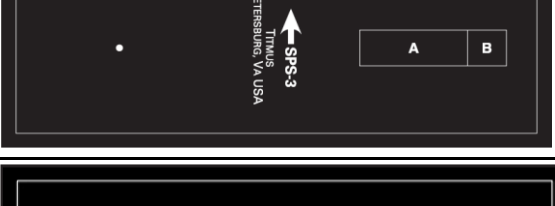
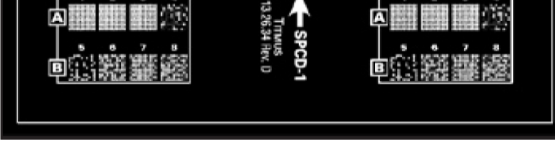


Titmus V3 Pediatric Model Vision Screener

Slide Package (SH # 52785)

| Slide | Slide Name | Testing Function | Testing Function Detail | Image |
|-------|-------------------------------------|--|---|--|
| 1 | 52-194 LEA SYMBOLS® | Binocularity & Acuity, Left & Right Eye | LEA SYMBOLS® to test Binocularity and Acuity, left & right eye. |  20/100 20/40 20/32 20/25 20/20 |
| 2 | SMPS-30-1 MI Preschool Acuity | Acuity, Right & Left Eye | Visual acuity testing using the Michigan Preschool Test. Right and Left Eye Images are different. Administer test at FAR. |  |
| 3 | SMPS-30-2 MI Preschool Acuity | Acuity, Right & Left Eye | Visual acuity testing using the Michigan Preschool Test. Right and Left Eye images are different. Administer test at FAR. |  |
| 4 | SMPS-30-3 MI Preschool Acuity | Acuity, Right & Left Eye | Visual acuity testing using the Michigan Preschool Test. Right and Left Eye images are different. Administer test at FAR. |  |
| 5 | SMPS-30-4 MI Preschool Acuity | Acuity, Right & Left Eye | Visual acuity testing using the Michigan Preschool Test. Right and Left Eye images are different. Administer test at FAR. |  |
| 6 | SAS-PT "E" Symbol Acuity | Acuity, Right & Left Eye | Tests visual acuity (sharpness of the eye). Right and Left Eye images are different. Administer test at FAR and using PLUS lenses. Refer to page 6 for Plus Lens Test. |  |
| 7 | SPS-3 Muscle Imbalance | Muscle Balance | This is a Muscle Balance Test to test Vertical and Lateral Phoria in children. For children that fail this test, it helps identify if there is a possibility of amblyopia (lazy eye) or strabismus (squint eyes). Test is administered first at far and then at near. |  |
| 8 | SPCD-1 Pediactric Color | Color Perception | Tests color perception. Tests uses photographially reproduced Titmus Pseudo-Isochromatic blocks. |  |