

# 12 Ways to use a LOGAN<sup>®</sup> ProxTalker<sup>®</sup>



## 1. Personal vertical picture schedule

- Use photos or picture symbols;
- Set up the morning schedule (seen here);
- Change the schedule for lunch, recess and afternoon.

1



## 2. Real object schedule

- Attach real objects to tags
- Record speech
- Play

2



## 3. Horizontal sequential task list

- Use with any age child for sequential tasks
- Try horizontal presentation (as shown)

3



## 4. Receptive language and vocabulary practice

- Use in conjunction with any classroom methods, curriculum & stories
- Can be used in conjunction with many individualized strategies

Find the picture of the “dog”

4



## 5. Expressive language for personal comfort and care requests

- Use in conjunction with paper and Velcro-based language training systems  
“I want a granola bar”
- Use the record tag for additional message or clarification  
“Chocolate covered, please.”



### More personal requests and communication

- Another example of a personal request  
“I want juice please.”
- Use the “speak all” green circular button to say the whole sentence at one time
- Spill the juice on the device? No problem!

5



## 6. Read math problems aloud

- Use the device familiar to the student to do math equations (seen here)
- Try using manipulatives to find the solution and hear ProxTalker<sup>®</sup> read it aloud

6



## 7. Read ingredients or steps for a recipe

- Cooking a meal or snack? No problem!
- Record the names of the ingredients or record the actions required to make a food

7



## 8. Teaching Braille letters, contractions and numbers

- Use the PROXTALKER® for students with vision challenges as well;
- Symbols, paired with recorded speech and Braille make powerful lessons for children with complex learning needs

8



## 9. Sentence scramble games

- Use the PROXTALKER® for sentence building (as shown)
- A sentence with four words can be read aloud using the “speak all” green circular tag

9



## 10. Spelling games for two, three and four letter words plus a checking space

- Each button can speak a letter
- What does that spell?
- Use the “speak all” green circular tag to speak all!

10



## 11. Sight word recognition

- Each button can speak a sight word
- A picture or symbol of an object can be shown & a child selects the correct word
- Use the “speak all” tag to string the letters or sounds together

11



## 12. Answering multiple choice questions

- Ask a question verbally and the PROXTALKER® user can find the answer
- Use PROXTALKER® across the curriculum in any class for any subject

12

If a student is working on commenting, keep the LOGAN® PROXTALKER® loaded with up to five recorded comment tags, e.g., “That’s cool,” “No way!” “Aw man!” or “I need a break.”

Change them often or leave familiar favorites on the LOGAN® PROXTALKER®