
## Quick Reference Guide – easyTymp™

### Powering the Unit

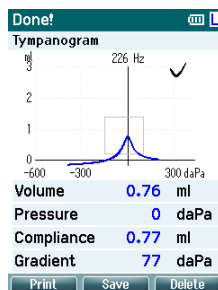
1. Press the **RED** or **BLUE ARROW**   to turn the unit **ON**.
2. The test will default to the protocol last selected.
3. Press and hold the **RED and BLUE ARROWS**   together to turn the unit **OFF**.

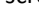
**NOTE:** If “show pass/fail” is turned on in the set-up, a  $\checkmark$  (check mark) indicates a pass, and an X indicates a refer result.

### Conducting a Test

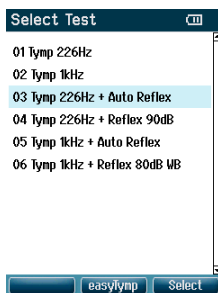1. Press **SELECT TEST**  on left side of screen.
2. Use **UP** and **DOWN ARROWS**   to highlight desired test protocol.

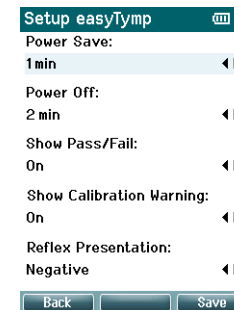
3. Press **SELECT** on the right side of screen.
4. Select test ear using the **RED (RIGHT)** or **BLUE (LEFT) ARROW** keys.  
5. Test will start when probe is placed in the ear. Status of the test is shown at the top of screen.
6. When test is finished, use **UP** or **DOWN ARROWS**   to scroll through screen and review results.

3. Place unit in cradle to download test results.

**NOTE:** If results do not download automatically, click **REQUEST MEASUREMENT** at top of software module screen.

### Set-Up Menu

1. Press **SELECT TEST** .
2. Press **EASYTYMP**.
3. Scroll with **UP** and **DOWN ARROWS**   to highlight a Set-up Menu selection.
4. Press **SELECT**.
5. Use **UP** and **DOWN ARROWS**   to scroll.
6. Press **RIGHT** or **LEFT ARROWS** to change selection.  

7. Press **SAVE** to save all changes and exit back to Set-up Menu screen.
8. Press **BACK** to exit back to protocol list.

### Printing Test Results to Thermal Printer

1. To print results on the built-in thermal printer, place the unit in the cradle. Results should print automatically.

**NOTE:** If results do not print automatically, press the **PRINT** button. (Manual or automatic printing can be selected in the set-up menu.)

### Saving Test Results to NOAH Module

1. Make sure cradle is connected to computer via USB cord.
2. Open MAICO Impedance module via NOAH database.

**NOTE:** Upon initial set-up of the software, click on **EXTRAS** and select **SETTINGS**. Select **USB (easyTymp Communicator)**. Then click **OK**.

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