



Hillrom™

Welch Allyn®
RetinaVue® 700 Imager
Startup Guide

To get started, go to www.retinavue.com/700 for information about using and servicing your product >>

- How To Videos
- Directions For Use
- Service and Support
- Warranty Registration



1. RetinaVue 700 Imager	5. Lens cloth
2. USB cable	6. Wrist strap
3. Wall adapter plug and AC power cord	7. Directions for use
4. Lithium-ion battery	8. Anterior Imaging Spacer

Welch Allyn Technical Support
www.welchallyn.com/support

Mat. No. 728140, 80024198 Ver. B
Revision date: 2019-06
©2019 Welch Allyn, Inc.



(91)728140

1

Set up your camera.
Select whether your camera will be used with USB or Wireless.

RetinaVue Network – Wi-Fi

RetinaVue Network – USB

Next

2

Available Wi-Fi networks

aaWi-Fi

bbWi-Fi

ccWi-Fi

ddWi-Fi

Device setup
Select a Wi-Fi network

Info

3

Enter the passphrase for network "aaWi-Fi":

Passphrase

Remember this connection

Connect

4

https://www.retinavue.net/RN_CustomerPortal/

Launch an internet browser and enter the following address:
https://www.retinavue.net/RN_CustomerPortal/

5

Next

6

WelchAllyn
RetinaVue™ Network

Authorized users only. All activity on this website is monitored and recorded.

User Name:

Password:

Log In

7

Welcome to the RetinaVue Customer Portal

What would you like to do?

- Schedule a Patient (wireless only)
- View Exam Status
- View New Diagnostic Reports
- Search Diagnostic Reports
- View Statistics
- Manage Devices
- Manage Clinics
- Edit Company Information

Next

8

Add Device

Device Name	Serial Number	Device Access Code	Registration Date	Status	Detail
No devices are currently authorized for clinic(s) that you can manage.					

Back

9

Next

Enter the code on the RetinaVue Network Customer Portal to complete setup.

XXXX-XXXX

Do not power off the camera during registration.

Next

10

Device Registration

Device Registration Code

XXXX-XXXX

Enter

The device registration code can be found on the device's installation screen

11

Add New Device

Device Name
Mountain View And Valley View Clinics

Device Serial Number
34160044

Device Access Code
####

Select clinics to associate with this camera

Clinic Name	Address
Mountain View Clinic	1234 Red River Road, Canyon Top, UT
Valley View Clinic	4321 Bend Rd, Canyon Bottom, UT

Back

Add Device

12






Camera registered successfully.

Device setup
Registration

Next


Battery assembly and charging

1 

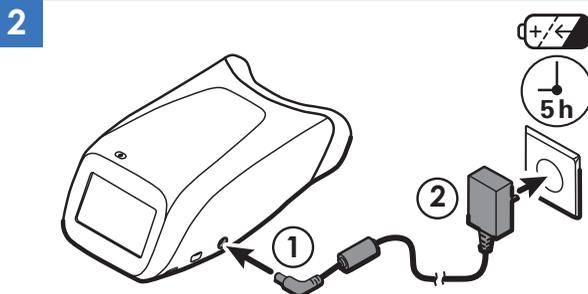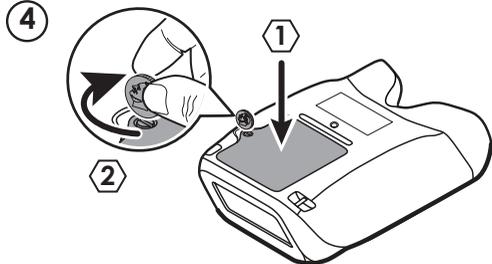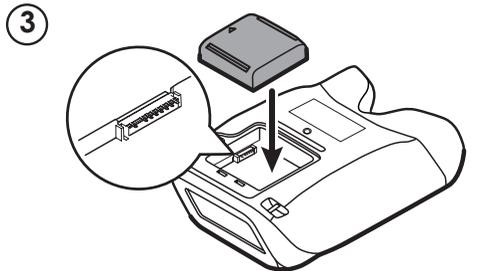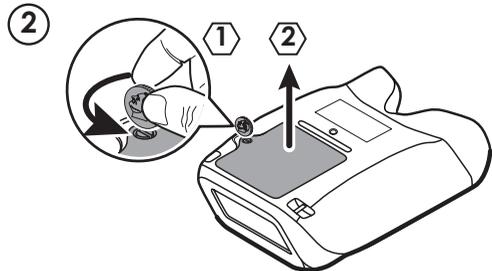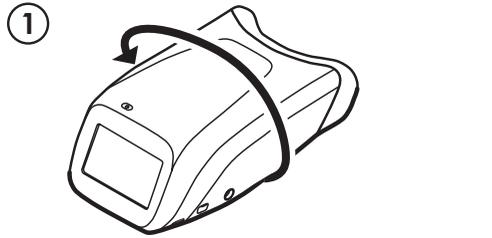


Startup imager

1

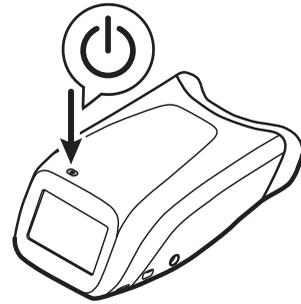


2

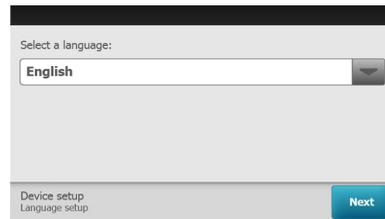

3



4

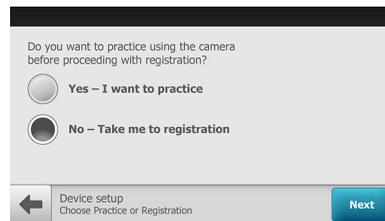

RetinaVue Network registration for USB or Wi-Fi connectivity



Call: 1.865.622.8380 to set up an initial account with the RetinaVue Network.



You will receive an email via Adobe's eSign service that includes your RetinaVue Professional Service Agreement.



Electronically sign and submit your RetinaVue Professional Services Agreement.



You will receive an email from RetinaVue Network System with the initial username and password and a link to the RetinaVue Customer Portal.



Go to the RetinaVue Customer Portal and complete your RetinaVue Network account registration.

For Wi-Fi connection, go to the next page.

Additional registration steps for USB connectivity



Upon completion of your account registration, download the RetinaVue Software from the RetinaVue Customer Portal onto the PC you will be using to transfer images to the RetinaVue Network.




Launch the RetinaVue Network software. Enter your RetinaVue Network Software Activation key into the RetinaVue Network software.

USB connection

1



2

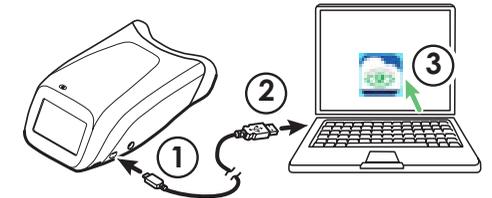

3

